



The easy durable way to stay dry™

## Instructions for Use

Did you know you can keep your cast or bandage dry while showering or bathing? You can with Seal-Tight Freedom...

*The easy durable way to stay dry.*

Here's how easy it is!

### Instructions for Use


1. After removing your cast protector from the box, unfold completely.
2. Pull SEAL-TIGHT® Freedom over cast by stretching diaphragm opening with thumbs as you hold the rubber diaphragm back.  
DO NOT apply by pulling on rubber diaphragm.
3. Push the diaphragm toward the cast so the inner edge of rubber points up and away from cast.
4. To Remove: Dry rubber diaphragm and stretch diaphragm to ease over heel or elbow.  
DO NOT remove by pulling on the bag.

For therapeutic care/daily showering or bathing only. When submersing, do not put weight on plastic ring. Not intended for recreational water activities. Use care around water, may be slippery when wet. With children, adult supervision recommended. Consult your doctor if diaphragm causes discomfort or swelling.

Brownmed warrants this product to be free of manufacturing defects. Intended for use on single patient.



Scan for product demo

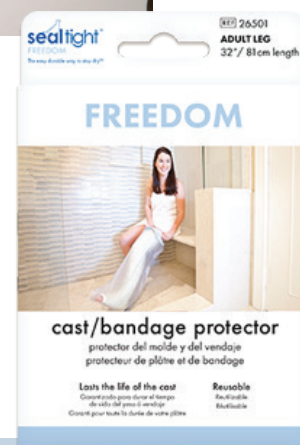

www.brownmed.com

#### Executive:

280 Summer Street  
Suite 400  
Boston, MA 02210  
P 857-317-3354  
F 857-317-3355

#### Sales:

4435 Main Street  
Suite 820  
Kansas City, MO 64111  
TF 877-853-5518  
P 816-581-7001  
F 816-581-7005

#### Manufacturing:

1300 Lundberg Drive West  
Spirit Lake, IA 51360  
TF 800-843-4395  
P 712-336-4395  
F 712-336-2874

©2016 Brownmed - SEAL-TIGHT, the SEAL-TIGHT logo, "The easy durable way to stay dry" and "Stay dry while getting wet" are trademarks or registered trademarks of Brownmed.