



## Instructions for Use

Did you know you can keep your cast or bandage dry while showering or bathing? You can with Seal-Tight Sport...

*The easy durable way to stay dry.*

Here's how easy it is!

### Instructions for Use

1. After removing your cast protector from the box, unfold completely.
2. Pull SEAL-TIGHT® Sport over cast by stretching diaphragm opening with thumbs as you hold the diaphragm open. DO NOT apply by pulling on rubber diaphragm.
3. Push the rubber diaphragm toward the cast so the inner edge of rubber points up and away from cast.
4. To Remove: Dry rubber diaphragm and stretch diaphragm to ease over heel or elbow. DO NOT remove by pulling on the bag.

For therapeutic care/daily showering or bathing only. When submersing, do not put weight on plastic ring. Not intended for recreational water activities. Use care around water, may be slippery when wet. With children, adult supervision recommended. Consult your doctor if diaphragm causes discomfort or swelling.

Brownmed warrants this product to be free of manufacturing defects. Intended for use on single patient.



Scan for product demo



**cast/bandage protector**  
 protector del molde y del vendaje  
 protecteur de plâtre et de bandage

**Safe to Use for Swimming**  
 Seguro de usar para natación

**Reusable**  
 Reutilizable



www.brownmed.com

**Executive:**  
 280 Summer Street  
 Suite 400  
 Boston, MA 02210  
**P** 857-317-3354  
**F** 857-317-3355

**Sales:**  
 4435 Main Street  
 Suite 820  
 Kansas City, MO 64111  
**TF** 877-853-5518  
**P** 816-581-7001  
**F** 816-581-7005

**Manufacturing:**  
 1300 Lundberg Drive West  
 Spirit Lake, IA 51360  
**TF** 800-843-4395  
**P** 712-336-4395  
**F** 712-336-2874

©2016 Brownmed - SEAL-TIGHT, the SEAL-TIGHT logo, "The easy durable way to stay dry" and "Stay dry while getting wet" are trademarks or registered trademarks of Brownmed.