

BLUETOOTH® APPLICATION END-USER TUTORIAL

To operate the Body Furnace with Bluetooth®, you will need to use the Body Furnace app. You can download this from the Apple App Store or the Google Play App Store.

APP COMPATIBILITY

- Apple® devices running iOS 9 or newer (iPhone® 5, iPod touch® 5th Gen., or newer versions of these devices)
- Leading Android[™] devices

QUICK START INSTRUCTIONS:

- Download and install the Body Furnace application at the Apple App Store for your iOS device or the Google Play App Store for your Android device.
- 2. Put Body Furnace into Bluetooth mode by holding the control button for 5 seconds while the unit is powered off.
- 3. Open the Body Furnace App and follow the on-screen instructions to pair a new device or connect to an existing device.
- 4. Use the on-screen controls to adjust temperature and monitor battery levels.

ON-SCREEN CONTROL OVERVIEW



Start-up Screen



Searching For Known Devices



No Paired Devices Found



Bluetooth Pairing Setup



Pairing New Device



Successful Pairing



Pairing Failed



Connection Lost



Battery Status



Low Battery Warning



Control Panel



Help Menu



Settings Menu



Unit Settings



Auto-off