



The easy durable way to stay dry™

Instructions for Use

Did you know you can keep your cast or bandage dry while showering or bathing? You can with Seal-Tight Original...

The easy durable way to stay dry.

Here's how easy it is!

Instructions for Use

1. After removing your cast protector from the box, unfold completely.
2. Pull SEAL-TIGHT® Original over cast by stretching diaphragm opening with thumbs as you hold ring. DO NOT apply by pulling on rubber diaphragm.
3. Push the plastic ring toward the cast so the inner edge of rubber points up and away from cast.
4. To Remove: Dry rubber diaphragm and stretch diaphragm to ease over heel or elbow. DO NOT remove by pulling on the bag.

For therapeutic care/daily showering or bathing only. When submersing, do not put weight on plastic ring. Not intended for recreational water activities. Use care around water, may be slippery when wet. With children, adult supervision recommended. Consult your doctor if diaphragm causes discomfort or swelling.

Brownmed warrants this product to be free of manufacturing defects. Intended for use on single patient.



Scan for product demo



www.brownmed.com

Executive

101 Federal Street
29th Floor
Boston, MA 02110
P 857-317-3354
F 857-317-3355

Sales

4435 Main Street, Suite 820
Kansas City, MO 64111
TF 877-853-5518
P 816-581-7001
F 816-581-7005

Manufacturing

1300 Lundberg Drive West
Spirit Lake, IA 51360
P 712-336-4395
F 712-336-2874

©2016 Brownmed - SEAL-TIGHT, the SEAL-TIGHT logo, "The easy durable way to stay dry" and "Stay dry while getting wet" are trademarks or registered trademarks of Brownmed.

2016-21486 Rev. 0