



Instructions for Use

Did you know you can keep your cast or bandage dry while showering or bathing? You can with Seal-Tight Sport...

The easy durable way to stay dry.

Here's how easy it is!

1. After removing your cast protector from the box, unfold completely.
2. Pull SEAL-TIGHT® Sport over cast by stretching the opening of the cast protector over your cast while pulling up. You will know it's at the right position on your arm or leg when the seal opening sits comfortably above the top cast edge.

Important note: The seal should be tight, after all, it's keeping water off of your cast. Never trim with scissors or puncture. It's ok if the bag appears oversized, as this allows room for large casts and easy movement.

3. Being careful not to submerge your leg past the seal, insert protector into the water while simultaneously holding a portion of the seal away from your skin with one finger. This will allow any excess air to escape easily from the bag.
4. When the seal is approximately one inch from the water's edge, remove your finger so the seal fits snugly against your skin.
5. To Remove: Dry the protector thoroughly before stretching the seal over cast and remove. **DO NOT** remove by pulling on the bag.

For therapeutic care/daily showering, bathing or swimming. Use care around water, may be slippery when wet. With children, adult supervision recommended. Consult your doctor to be sure swimming is appropriate in your situation.

Brownmed warrants this product to be free of manufacturing defects. Intended for use on single patient.



Scan for product demo



www.brownmed.com

Executive

101 Federal Street
29th Floor
Boston, MA 02110
P 857-317-3354
F 857-317-3355

Sales

4435 Main Street, Suite 820
Kansas City, MO 64111
TF 877-853-5518
P 816-581-7001
F 816-581-7005

Manufacturing

1300 Lundberg Drive West
Spirit Lake, IA 51360
P 712-336-4395
F 712-336-2874

©2017 Brownmed - SEAL-TIGHT, the SEAL-TIGHT logo, "The easy durable way to stay dry" and "Stay dry while getting wet" are trademarks or registered trademarks of Brownmed.

2016-21485 Rev.0